For further assistance on how to use your Tenovi device, please contact a Tenovi representative at:

Phone: (714) 418-5658

**Email:** 

support@tenovi.com





## © 2021 Tenovi Co.

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.

Revision 05.01.2021




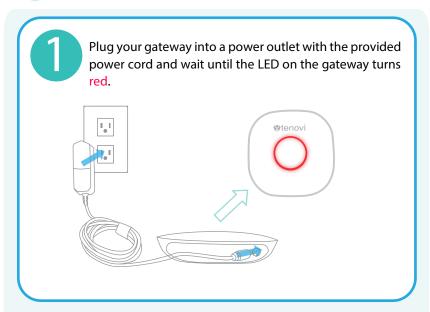

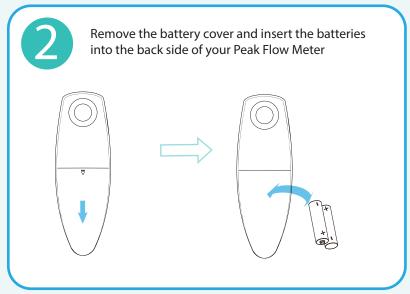



Press the power button for 3 seconds and wait for the Tenovi Gateway to start flashing yellow.

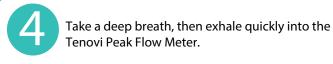

**Power Button** 



After the readings become visible on the screen, the Peak Flow Meter will transmit the data to the Gateway. If the LED ring is Green, you have properly taken a measurement for the day and that measurement has been successfully transmitted.

The Peak Flow Meter will automatically shut down.



Take one measurement once per day unless otherwise advised by your healthcare provider. The LED on your Tenovi device will turn Red at the beginning of each day indicating you must take measurement for that day. The LED ring will turn Green when you successfully take a measurement and transmit it to your healthcare provider.



Measurement Needed

Measurement Successful