

6 Take one measurement once per day unless otherwise advised by your healthcare provider. The LED on your Tenovi device will turn **Red** at the beginning of each day indicating you must take measurement for that day. The LED ring will turn **Green** when you successfully take a measurement and transmit it to your healthcare provider.



Measurement Needed



Measurement Successful



Warning

This device was issued to you from your doctor.
You must wait for a phone call for your assigned care manager before setting up this device.




© 2021 Tenovi Co.

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to

Tenovi, Co. unless otherwise noted.


Revision 05.01.2021

1 Plug your gateway into a power outlet with the provided power cord and wait until the LED on the gateway turns **red**.

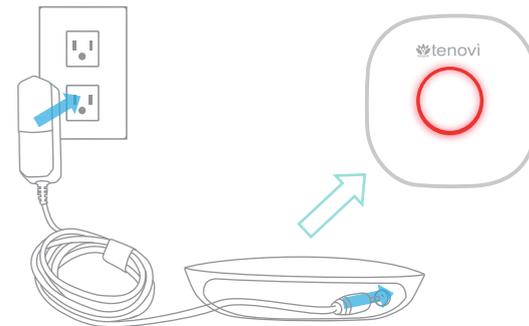


2

Insert batteries into the front of the thermometer. Stand within 10 feet of the gateway and press the blue button on the thermometer. The gateway will beep and the LED light will spin yellow, indicating that it has connected successfully to the thermometer.

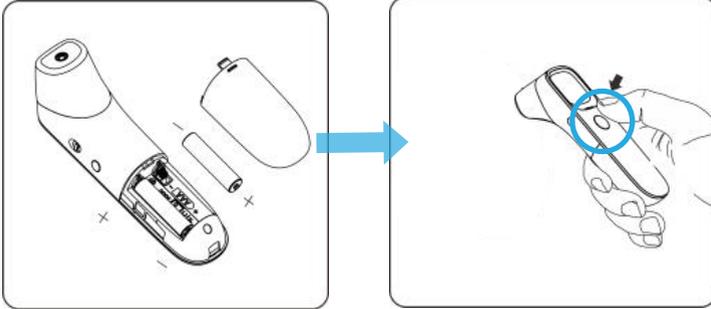


4

Press the "mode" button on the right side of the Thermometer. Make sure that the display symbol  is on the screen.

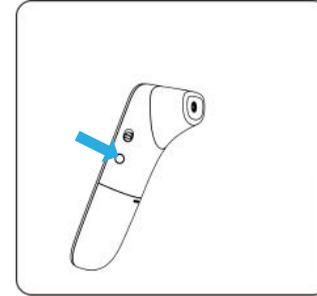

3

Ensure the cover is on the probe of the thermometer and press the power button.

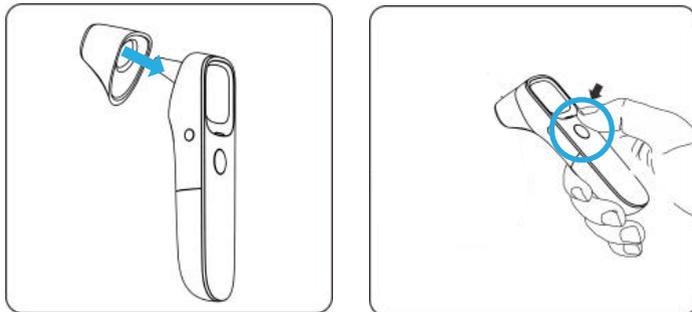

5

Stand within 10 feet of the Gateway and point the thermometer to the center of your forehead and press the blue button to take a measurement.

