


Pillbox Quick Start Guide

1

Plug in gateway


Plug your gateway into a power outlet with the provided power cord and wait until the LED on the gateway turns RED

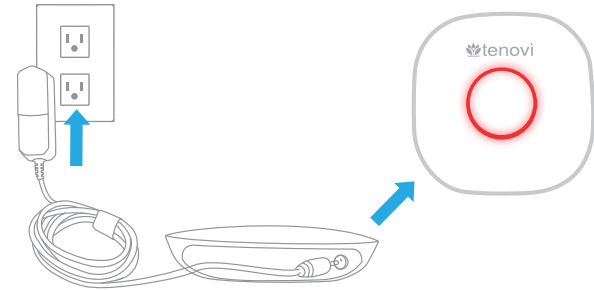

2

Power your gateway

Remove the paper pull tab from the battery compartment.

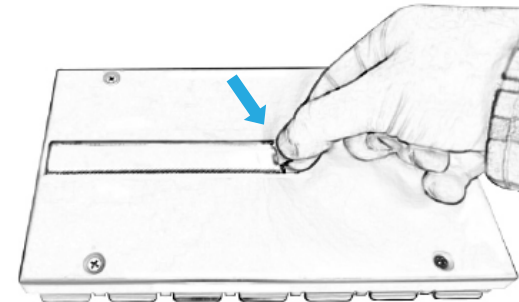


© 2021 Tenovi Co.
All rights reserved. Republication, duplication or modification
in whole or in part is prohibited without prior written permission
from Tenovi. All trademarks are owned by or licensed to
Tenovi, Co. unless otherwise noted.
Revision 05.01.2021

3

Refill your pillbox


Press the Refill button at the top of the pillbox to enter Refill Mode -- the compartments will start glowing. Fill your AM and PM pills in the respective AM and PM compartments by pressing down on each compartment button.

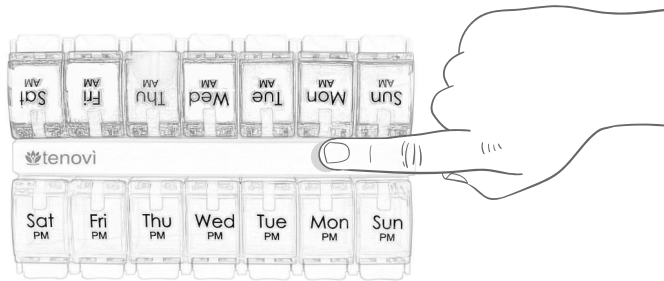

4

Exit refill mode


Press down on the refill button again to exit Refill Mode. You must enter Refill Mode each week when you refill your pillbox with medication.

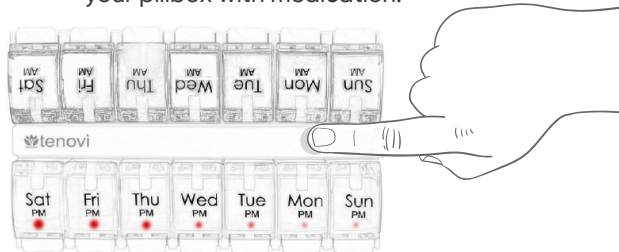

5

Take your medication

Take your daily medication as directed by your doctor by pressing down on the respective AM or PM compartment. The compartment corresponding to the current day and time will illuminate when you open any compartment.

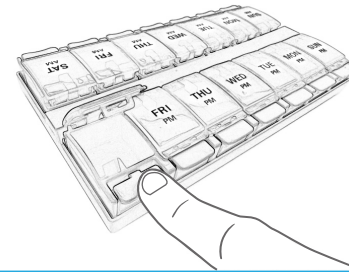

6

Use your pillbox daily

The Tenovi Pillbox will save all open events, even if you are away from the Tenovi Gateway. Whenever you open a pill compartment within 30 feet of the Tenovi Gateway, all stored open events will be sent to your care provider through the Tenovi Gateway. Your Gateway will turn from red to green and will beep if a medication event was sent that day.



Measurement Needed



Measurement Successful