

1

Power Your Gateway

Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns **RED**.



For further assistance, please visit [Tenovi.com/support](https://www.tenovi.com/support).


tenovi

© 2025 Tenovi Co.
All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.
Revision 05.01.2024

2

Insert Batteries

Insert batteries into the back of the Peak Flow Meter.

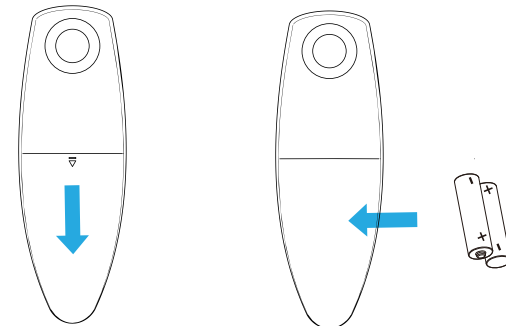

Connect To Gateway

3


Stand within 20 feet of the Tenovi Gateway, press the power button for 3 seconds and wait for the Tenovi Gateway to start flashing yellow.



Displaying Measurement

5


Your measurement will be displayed on the Peak Flow Meter screen and the Gateway LED should turn green. Press the power button and start at Step 3 if you need to take another reading.

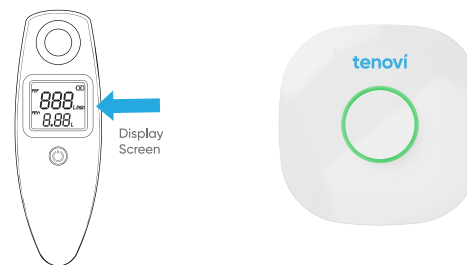

Take A Measurement

4

Take a deep breath and then exhale quickly into the Tenovi Peak Flow Meter.

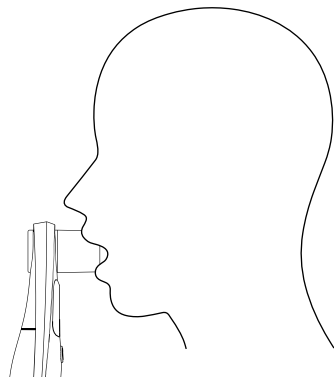

Daily Measurements

6

Take one measurement each day unless advised otherwise by your healthcare provider. The LED will turn RED each morning and then turn GREEN once a measurement has been successfully transmitted.



Measurement Needed

Measurement Successful