

tenovi


Blood Glucose Monitoring System

Quick Start Guide


1. Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns red.




2. Wash your hands with soap and water, then dry them thoroughly with a clean towel.




3. Twist off the cap of the lancet device.

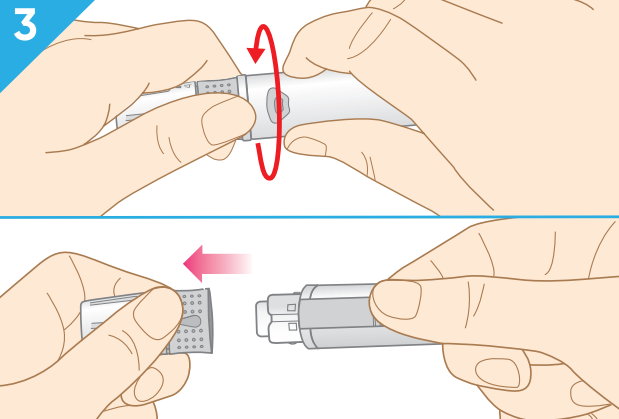


4. Insert lancet into device.

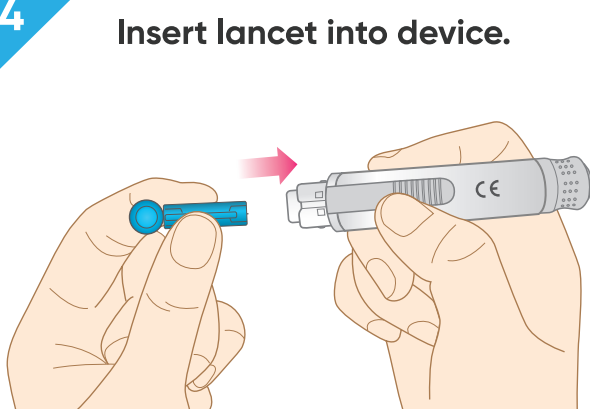


5. Twist off the lancet cap.

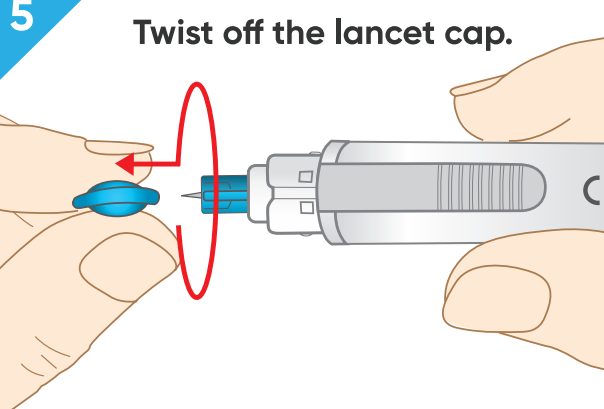


6. Twist the device to activate the lancet.

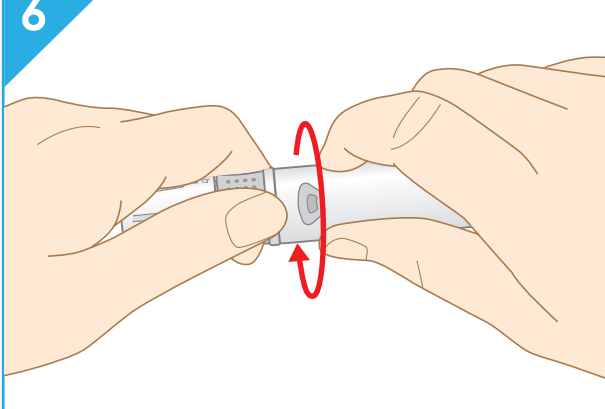


7. The lancet should be extended. A blue checkmark indicates correct activation, and a red X indicates incorrect activation.

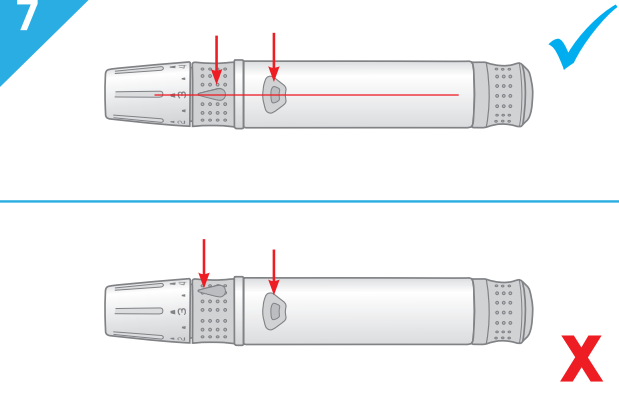


8. Twist the device to activate the lancet.

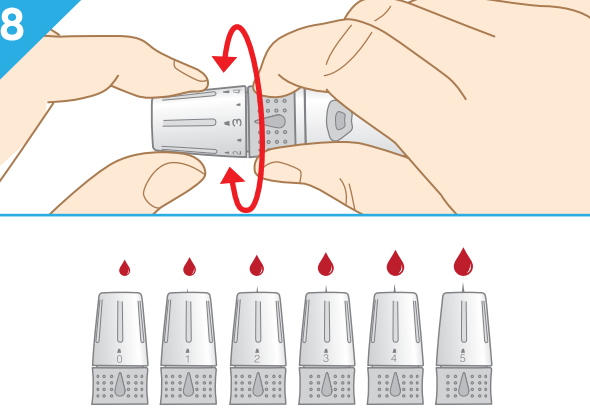


9. Pull back on the lancet device end to prep for insertion.

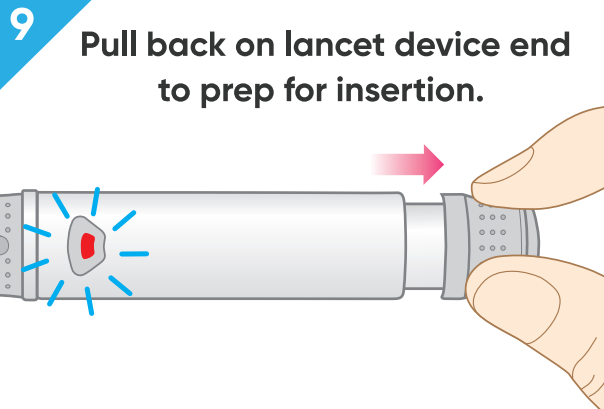

10. Insert the test strip into the device.

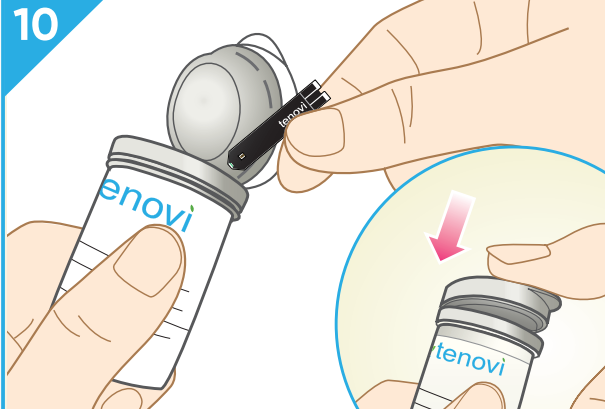


11. Insert the test strip fork side up to turn the device on.

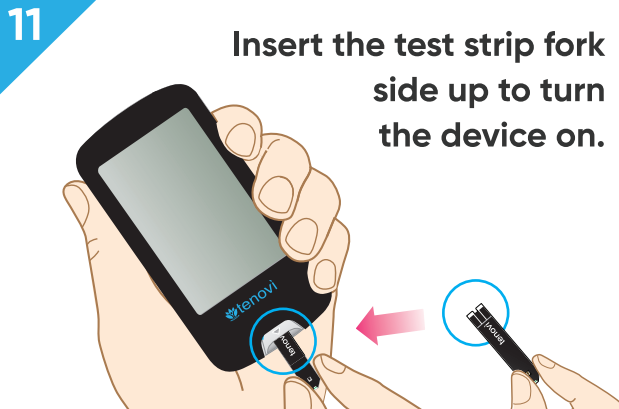


Blood Glucose Monitoring System – Quick Start Guide

12

The blood drop on the screen indicates the device is ready to take a reading.

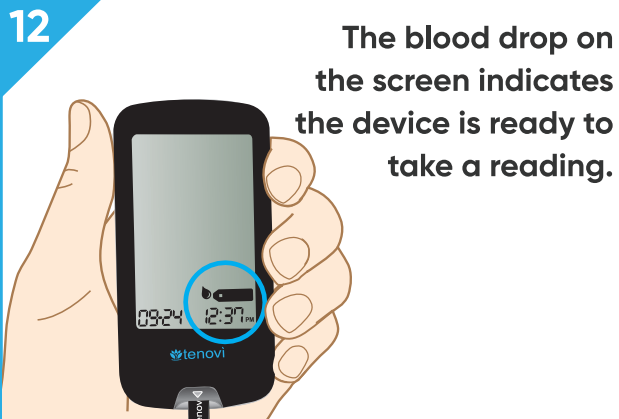An illustration of a hand holding a black Tenovi device. The screen shows a blood drop icon and the time 12:37. A blue circle highlights the blood drop icon.

13

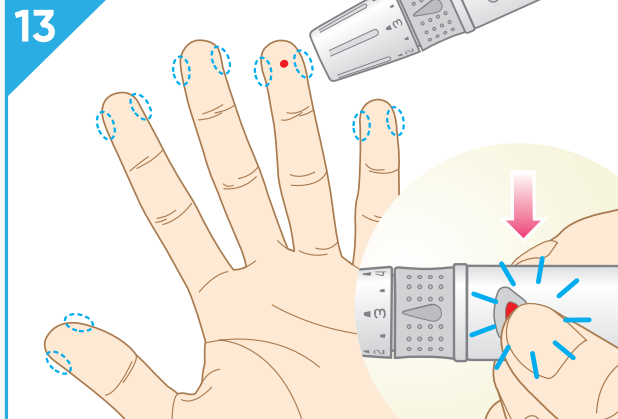
An illustration of a hand with a lancet pricking the index finger. A red arrow points down to the lancet tip. Blue dashed circles are around the other fingers.

14

Place the end of the test strip into the drop of blood. Hold it steady.

An illustration of a hand holding a black Tenovi device. A test strip is inserted into the device, and a blood drop is on the screen.

15

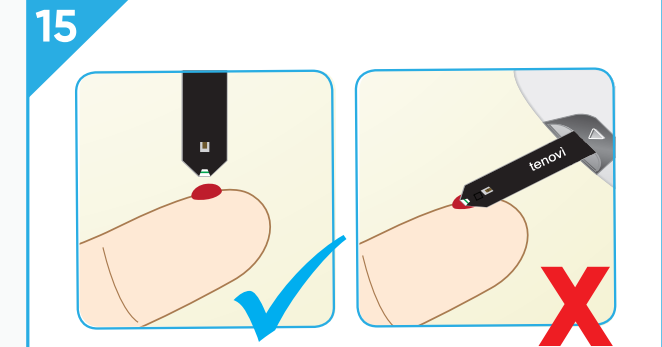An illustration showing two ways to insert a test strip. The left side shows the strip being inserted from above, marked with a blue checkmark. The right side shows the strip being laid flat, marked with a red X.

Insert from above, do not lay flat.

16


The device will count down from 5.

An illustration of a hand holding a black Tenovi device. The screen shows a large number '5' inside a dashed blue circle. Below the screen is a digital timer showing 00:00:05. A blue arrow points from the device to the timer.

00:00:05


17

Your reading will be displayed.

An illustration of a hand holding a black Tenovi device. The screen displays a glucose reading of 92 mg/dL. A blue circle highlights the reading.

18

Remove the test strip to automatically send data to your gateway

An illustration of a hand holding a black Tenovi device. A test strip is being removed from the device.

19

Throw away your test strip and lancet into a sharps container.

An illustration of a hand disposing of a test strip and lancet into a red sharps container with a biohazard symbol.

tenovi

© 2025 Tenovi Co.
All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.
Revision 05.01.2024