


1

Remove Pull Tab

Remove pull tab from the battery compartment to power on the device.

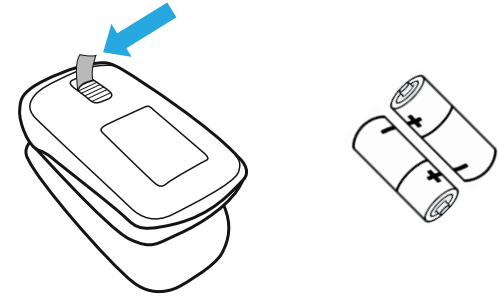

For further assistance, please visit [Tenovi.com/support](https://www.tenovi.com/support).


tenovi

© 2025 Tenovi Co.
All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.
Revision 05.01.2024

2

Insert Finger Into Pulse Ox


Insert one finger completely into the measurement section of the Oximeter with your fingernail **pointing up** towards the screen.

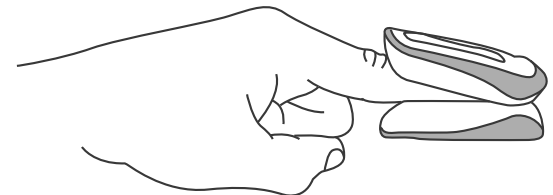

Press Button

3


Press the **power button** on the oximeter. Keep your finger in the oximeter for 20 seconds

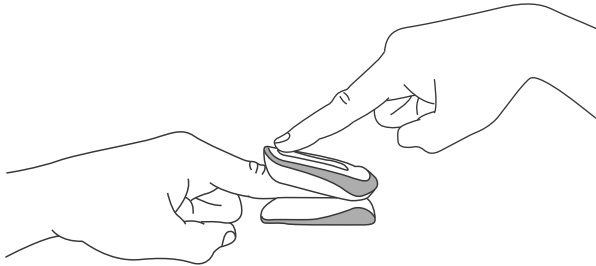

Wait 20 seconds

4

Once the measurement has been completed, the device will then lock the reading and begin to transmit the data over the cell network.

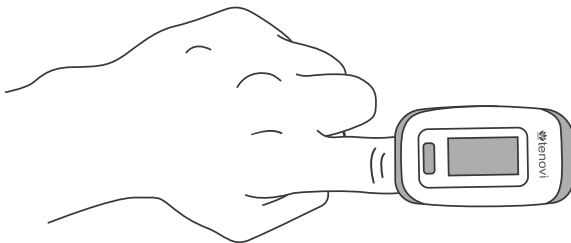

Measurement Taken

5

"**Success**" will display when the reading has been transmitted. When "**FAILED**" is shown, the reading will be saved and the device will try and send again on your next measurement.

Success

For further assistance, please visit [Tenovi.com/support](https://www.tenovi.com/support).

tenovi