

tenovi

1

Power Your Gateway


Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns **RED**.



2

Insert Batteries

Insert batteries into the back of the thermometer. Press the button on the front to turn the device on.

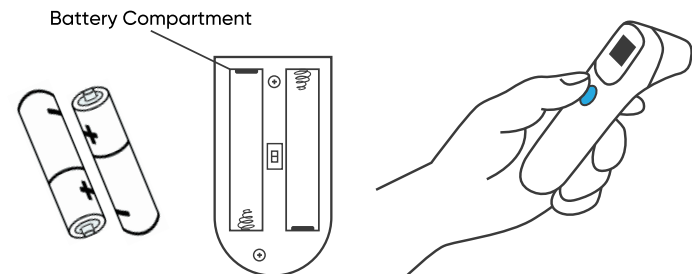


3

Take A Measurement

Point the thermometer at the center of the forehead. **Press the button** to take a reading.


To take in-ear measurements, consult your care provider

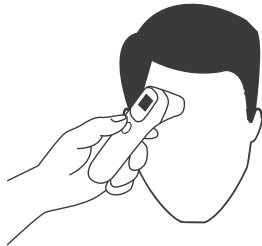

5

Cleaning

- Clean forehead before use
- Clean the thermometer after use
- Wait 5-10 minutes after cleaning device to use it

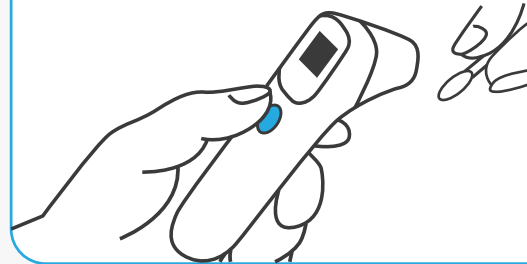

4

Daily Measurements

Take one measurement each day unless advised otherwise by your healthcare provider. The LED will turn **RED** each morning and then turn **GREEN** once a measurement has been successfully transmitted.



Measurement Needed



Measurement Successful

For further assistance, please visit Tenovi.com/support.

tenovi

© 2025 Tenovi Co.
 All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Tenovi. All trademarks are owned by or licensed to Tenovi, Co. unless otherwise noted.
 Revision 05.01.2024